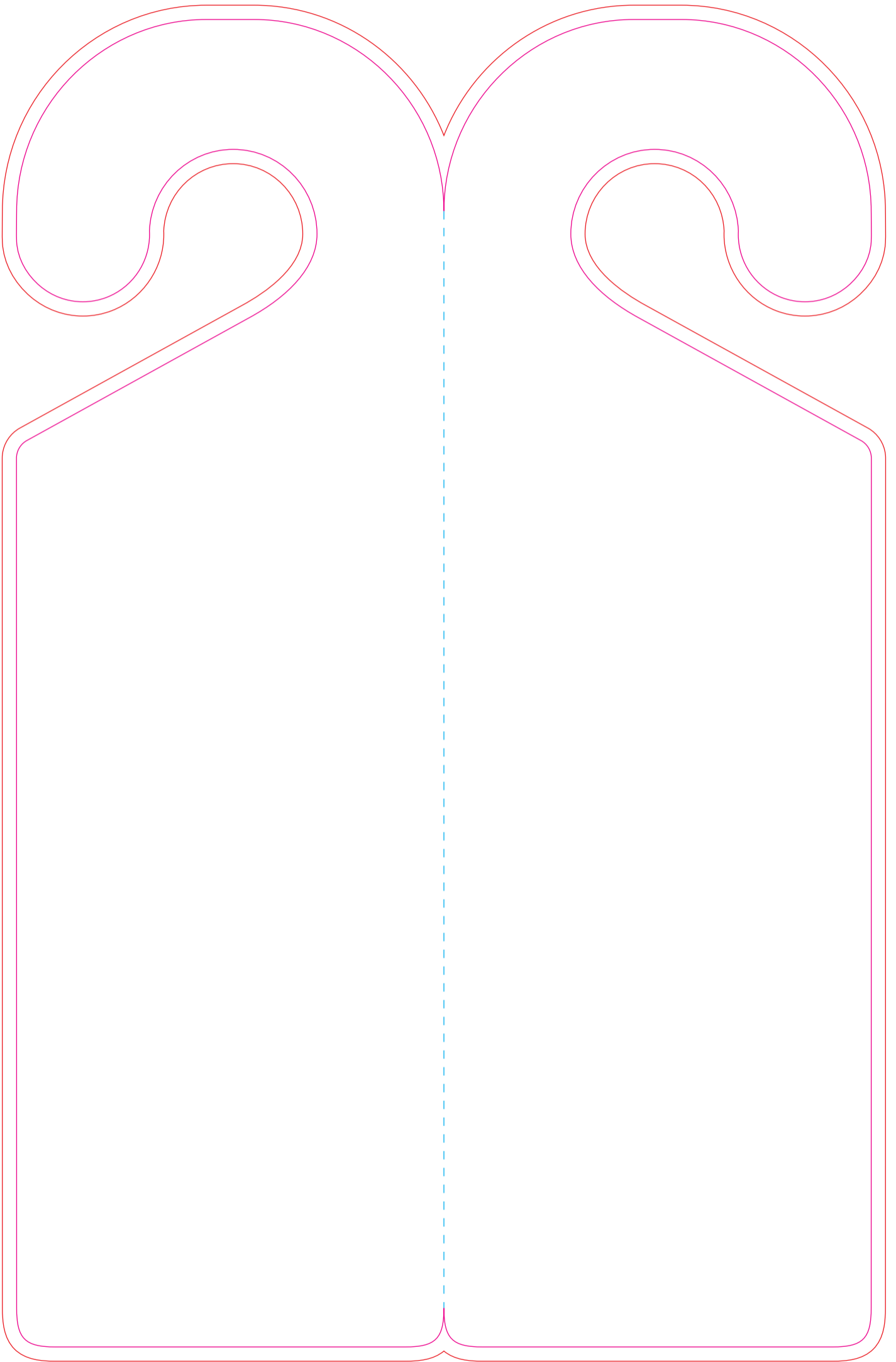
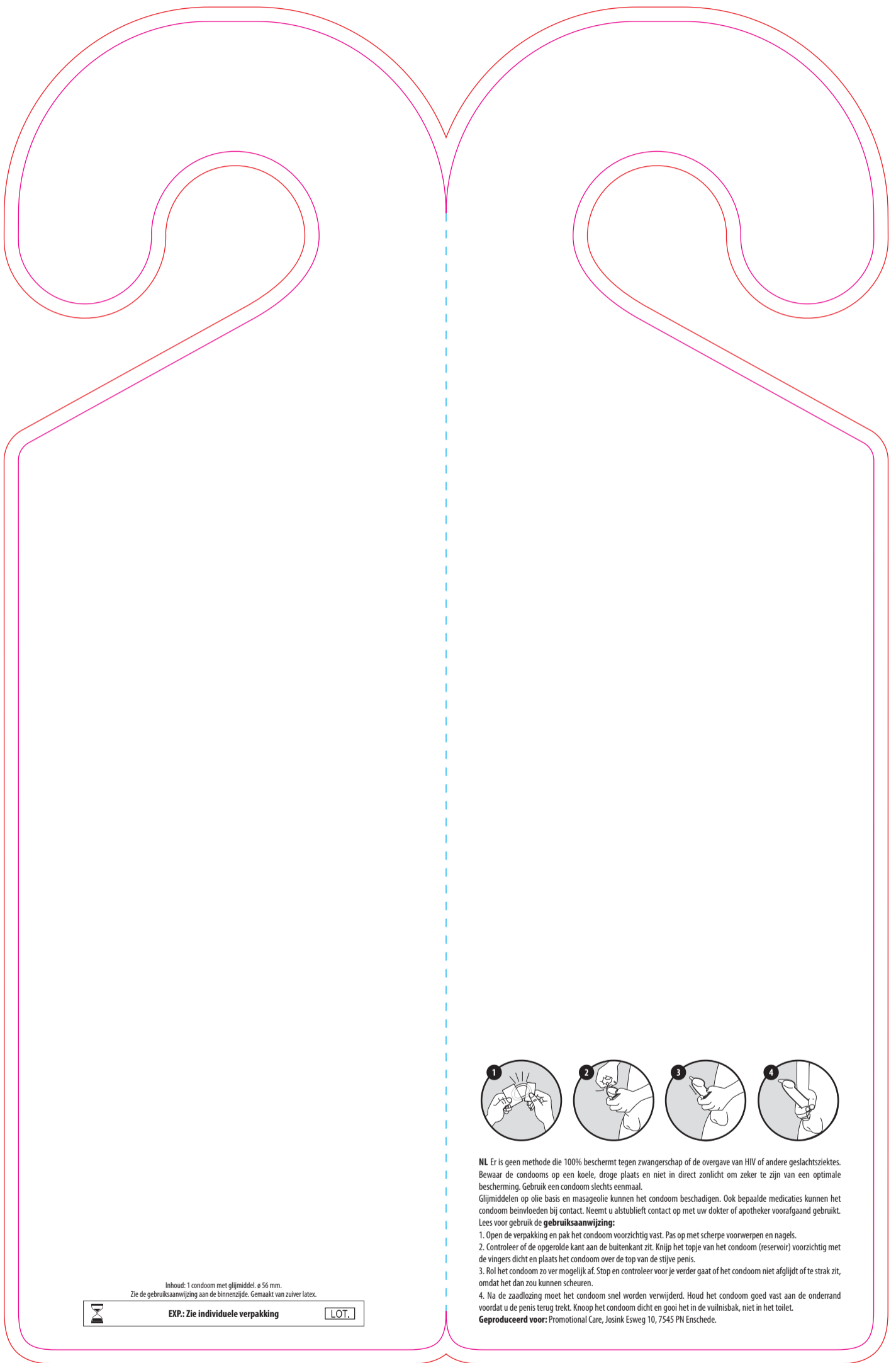
8321

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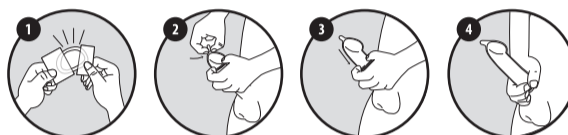
LOT.

## EN

**EN** There is no method of contraception can provide 100% protection against pregnancy or the transmission of HIV or other STD's. Store in a cool dry space away from direct for optimal protection. Only use a condom once.

Oil based lubricants or massage oil could damage the condom. Also some medications might affect condoms on contact. Please consult your doctor or pharmacist prior to use.

Read before use the **instructions**:

1. Open the foil wrapper carefully down one side to avoid tearing the condom. Be careful with sharp objects.
2. The condom can only easily roll on in one direction so you need to make sure that the rolled-up side is on the outside. Squeeze the end of the condom to get rid of any air and place over the head of the penis.
3. While squeezing the tip of the condom, roll it down over the length of the erect penis. You need to make sure the condom fits snugly over your penis with no baggy spots.
4. When the man has come, hold the condom in place and carefully withdraw the penis from your partner. Remove the condom while the penis is still hard. Close the condom and throw it in the trash can, not in the toilet.

**Produced for:** Promotional Care, Josink Esweg 10, 7545 PN Enschede, the Netherlands.

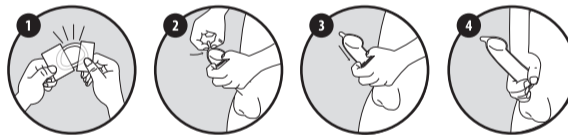
Content: 1 lubricated condom with reservoir, ø 56 mm.  
See instructions inside. Made of natural rubber latex.

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